IMPORTANT NOTICE
AVIS IMPORTANT
WICHTIGER HINWEIS
AVISO IMPORTANTE
BELANGRIJKE INFORMATIE
COMUNICAZIONE IMPORTANTE
VIKTIG MERKNAD
TÄRKEÄ ILMOITUS
VIKTIG INFORMATION
VIGTIG INFORMATION
AVISO IMPORTANTE
POMEMBNO OBVESTILO
DŮLEŽITÉ UPOZORNĚNÍ
FONTOS INFORMÁCIÓK
WAŻNA INFORMACJA
ÖNEMLİ DUYURU
DÔLEŽITÉ UPOZORNENIE
ΣΗΜΑΝΤΙΚΗ ΓΝΩΣΤΟΠΟΙΗΣΗ
ВАЖНОЕ ЗАМЕЧАНИЕ
AVISO IMPORTANTE
VAŽNA OBAVIJEST
VAŽNA NAPOMENA
ВАЖНА ИНФОРМАЦИЯ
AVIZ IMPORTANT
OLULINE MÄRKUS
MIKILVÆG TILKYNNING
SVARBI PASTABA
SVARĪGS PAZIŅOJUMS
NJOFTIM I RĒNDĒSISHĒM
ВАЖНО ИЗВЕСТУВАЊЕ
ВАЖЛИВЕ ЗАУВАЖЕННЯ
重要说明
중요 고지
留意事項
NOTIS PENTING
PEMBERITAHUAN PENTING

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ENGLISH

USING THE SET CHANGE REMINDER WITH THE MEDTRONIC EXTENDED INFUSION SET AND EXTENDED RESERVOIR

The Extended infusion set (all model types for MMT-431A, MMT-432A, MMT-441A, MMT-442A, MMT-443A) and Extended reservoir (all model types for MMT-342) can be used for up to seven days with the insulin pump models listed here.

The Set Change reminder alerts you when it is time to change your infusion set and reservoir. The Extended infusion set and Extended reservoir can be used with your pump for up to a maximum of seven days. The Set Change reminder can only be programmed for up to three days and should be turned off to avoid confusion. The Set Change reminder in the pump is set to off by default. If you have turned on this setting, follow these steps to turn it off:

MiniMed™ 780G Pump

1. Press  and then select .
Alert Settings > Reminders > Set Change
2. Select **Reminder** to turn the reminder off.
3. Select **Save**.

MiniMed™ 700, 720G, and 740G Pump

1. Press  and go to the Set Change screen.
Reminders > Set Change
2. Select **Reminder** to turn the reminder off.
3. Select **Save**.

MiniMed™ 770G, and 670G Pump

1. Press  and go to the Set Change screen.
Options > Reminders > Set Change
2. Select **Reminder** to turn the reminder off.
3. Select **Save**.

MiniMed™ 640G Pump

1. Press  and go to the Set Change screen.

Reminders > Set Change

2. Select **Reminder** to turn the reminder off.

3. Select **Save**.

Note: For warnings related to the Extended infusion set and the Extended reservoir, refer to the user guide provided with the devices.

Use only Medtronic infusion sets and reservoirs with your Medtronic insulin pump.

Contact a local Medtronic support representative for assistance.

© 2020 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

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© 2020 Medtronic. All rights reserved. Medtronic, Medtronic 로고 및 Further, Together는 Medtronic의 상표입니다. 기타 모든 브랜드는 Medtronic 기업의 상표입니다.
© 2020 Medtronic. All rights reserved. Medtronic、Medtronicロゴおよび Further, Togetherは、Medtronicの商標です。その他の製品名はすべて、メドトロニック社の商標です。
Medtronic B.V.