

INSTALL GUIDE

NEW CareLink™ Personal uploader

Upload to more easily
partner with your doctor

IMPROVE YOUR DOCTOR VISITS BY UPLOADING TO CARELINK™



Understand your glucose pattern



Upload regularly



Collaborate with your doctor

GET STARTED BY DOWNLOADING THE NEW UPLOADER INSTALL FILE


A one-time download per computer where you wish to upload your device.



- 1 Log in to CareLink™ Personal
 - Click "How to install Uploader"



- 2 Click the download link and select a location to save the uploader file

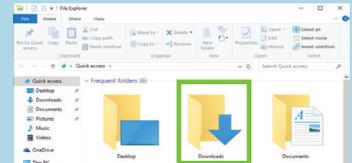


- 3 Find the downloaded file. Open the file and select "Run"

AFTER DOWNLOADING, INSTALL THE UPLOADER




- 1 Choose your language

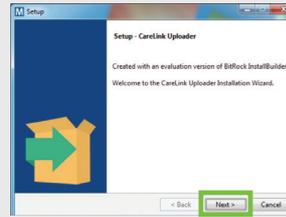

- 2 Click "Next"



- 3 Click "Next"

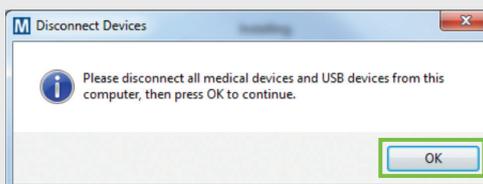

- 4
 - Make sure your CareLink™ USB or meter are not plugged in and click "OK"
 - Click "Finish" to complete the installation



NOW, YOU'RE READY TO UPLOAD

Return to your CareLink™ Personal Homepage as you are ready to upload your Pump or Meter to CareLink™.

Please turn over for upload instructions.

Medtronic