

UPLOAD GUIDE

NEW CareLink™ Personal uploader
Your first step to understanding more about your diabetes.

INITIAL FOUR-STEP PROCESS TO ADD A NEW PUMP OR METER

1 Choose pump or meter

Pump

Meter

2 Select the pump model

2 Select the meter brand if you want to upload BG meter data only

3 Enter pump serial number

3 Select meter model

4

- Make sure the pump is ready
- Click "Add Pump"

4

- Make sure the meter is ready
- Click "Add Meter"

TO ENSURE SUCCESS, GET EVERYTHING READY
Plug your USB or meter into your computer

- Place your pump as close as possible to the CareLink™ USB or meter that is plugged into your computer
- Keep the meter linked with the pump



Make sure your pump is ready to upload
Check that the pump is not on one of these screens.



- No temp basals
- No alarms
- No active bolus
- No low battery

TWO EASY STEPS TO UPLOAD PUMP OR METER DATA

STEP 1

Select the device

STEP 1

- Make sure the device is ready to upload
- Click "Upload Now"

Safety Information: CareLink™ software

CareLink software is intended for use as an adjunctive tool to help manage diabetes and is NOT intended to be relied upon by itself.

Country details here
Address line here

LINK WITH YOUR DOCTOR



Link to your doctor's CareLink™ system for more efficient data sharing

© 2018 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. 950M17620-011 201701001